


an-lens MD8



an-lens MD4



an-lens MD6

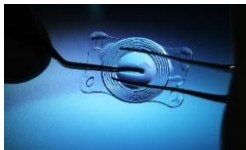


an-lens MH9



an-CTR




Instruments

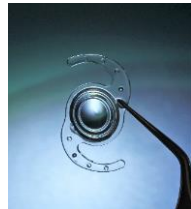

an-lens MD2



alexos3

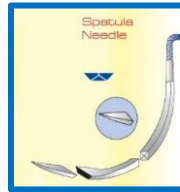

Sutures



an-blue




Viscoelastics



an-HyPro

an-Instructions

Loading Single Use CTR Injector

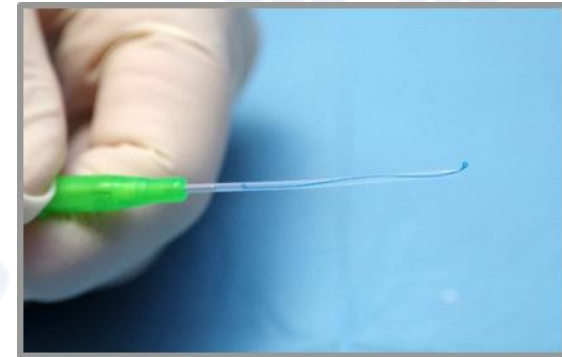

See all our **an-vision** products at:
www.an-vision.com or contact us!

Specifically Designed for Veterinary Use




1

Pull the plunger of the syringe-injector back a little bit.




2

Take the an-CTR with the holding forceps gently out of the case.

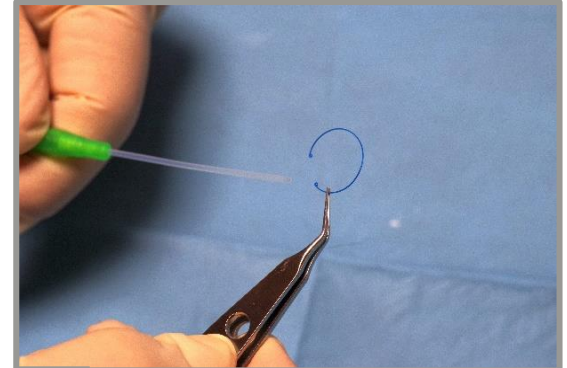


3

Insert the CTR into the tube of the syringe.

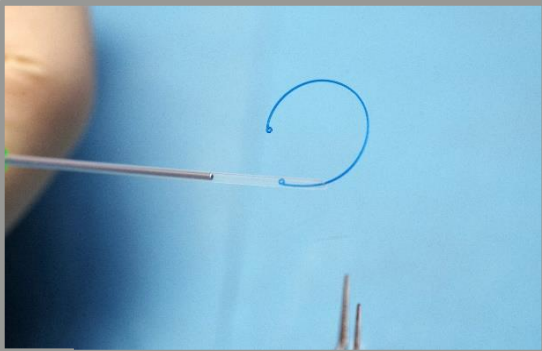

4

Slide the CTR in step by step while pulling back the plunger also step by step.

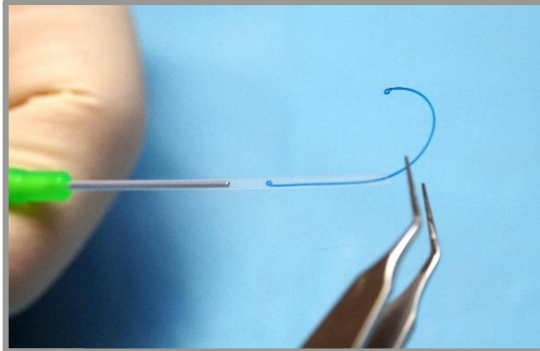

5

When the CTR is nearly fully inserted it is ready for implantation.



6

Injector with loaded CTR

an-vision Inc.
9067 S. 1300 W.
Salt Lake City, UT 84088 USA
phone: (801) 561-5040
fax: (801) 561-5393
www.an-vision.com



info@an-vision.com