


Preloaded Capsular Tension Ring (an-PTR) Helpful Tips


- 1 – When removing the device from the sterile packaging, **pull straight up on the injector** (or even better, pull the front part up first) to not break the tension ring off at the tip of the injector.
- 2 – There is a small hook on the end of the plunger that loops through an eyelet on the end of the tension ring. When you go to implant, **make sure the writing (the “R” and a clock-wise arrow) are pointed up** (towards the ceiling), and the tension ring will release easier from the eyelet (dropping your wrist will bring the hook anterior in the eye and gravity will help pull the eyelet down and off of the hook).
- 3 – See diagrams on reverse. Reach out with any questions!

an-vision
801-561-5040
info@an-vision.com


- 1 - Remove the PTR injection system from the sterile blister packaging as shown below (lifting the front up first)

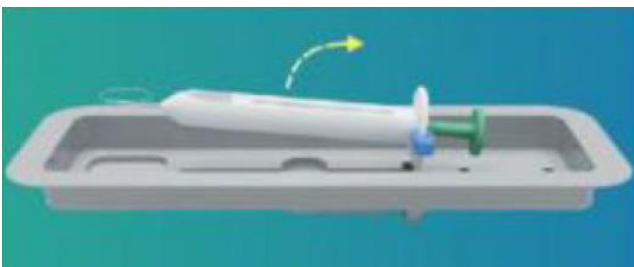

- 2 - Remove the blue plunger lock

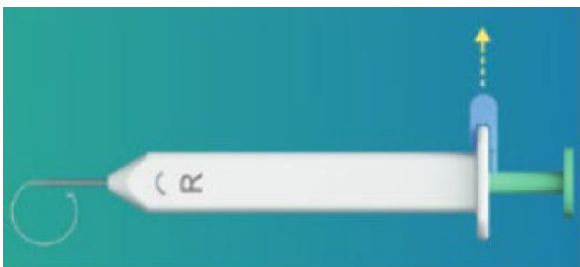

- 3 - Pull the plunger back to load the tension ring into the injector

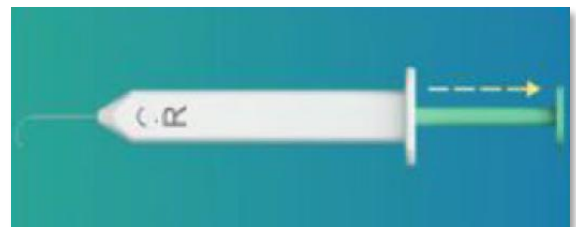

- 4 - Keeping the writing facing upwards, advance the plunger forward to implant the tension ring.

